



Canadian Sport Horse Association
PO Box 1625
Holland Landing, ON L9N 1P2
Tel: 905-830-9288 Fax: 905-830-9635

DNA Genetic Marker Test Kit

INSTRUCTIONS FOR OBTAINING AND SUBMITTING MANE HAIR SAMPLES FOR DNA TESTING

1. Pull hair samples from above the withers on the mane. **DO NOT CUT HAIR** as it will not provide DNA necessary for the test results.
2. Pull the hair evenly and directly out of the neck so that the hair does not break.
3. Pull 15 to 25 hairs with the root ball attached. This number of roots will not leave a hole in the mane.
4. On the reverse side of this page, place the hair in a straight line across the center area, which has been reserved for the hair. Keep the hair together with the roots to the left as indicated. Do not curl the hair. Tape the center of the hair shafts with a piece of tape to hold it in place.
5. Fold this sheet (mane hair kit) as you would a business letter. Do each sample one at a time and put each form into its individual envelope. Please ensure that hair from one horse only is included in each kit to prevent cross-examination of samples. Cross contamination of samples can confuse test results and the test may have to be repeated using another hair sample at an additional cost.
6. Enclose payment by cheque, Visa or MC and mail to:

Canadian Sport Horse Association
PO Box 1625
Holland Landing, ON L9N 1P2

DNA MANE HAIR TEST 46.73 + GST (7%) = \$50.00
or \$46.73 + HST (15%) = \$53.74

DO NOT SEND DNA TEST KITS TO MAXXAM OR CLRC

Visa _____ MasterCard _____ Number _____ / _____ / _____ / _____

Name: _____ Expiry Date ____ / ____
as shown on card

Signature _____ Date _____