



Canadian Sport Horse Association

Box 970, Richmond, Ontario, K0A 2Z0

Tel: 613-686-61618 • Fax: 613-686-6170

DNA GENETIC MARKER TEST INSTRUCTIONS

1. Pull hair samples from above the withers on the mane. DO NOT CUT HAIR as it will not provide the DNA necessary for the test results.
2. Pull the hair evenly and directly out of the neck so that the hair does not break.
3. Pull 15 to 25 hairs with the root ball attached. This number of roots will not leave a hole in the mane.
4. On the DNA Pulled Hair Sample form, place the hair in a straight line across the center area indicated. Keep the hair together with the roots to the left as indicated. Do not curl the hair. Tape the center of the hair shafts with a piece of tape to hold it in place.
5. Fold the sheet (DNA Pulled Hair Sample) as you would a business letter. Do each sample one at a time and pt each form into its individual envelope. Please ensure that hair from one horse only is included in each kit to prevent cross-contamination. Cross contamination of samples can confuse test results and the test may have to be repeated using another hair sample at an additional cost.
6. Enclose payment by cheque, VISA or MasterCard and mail to:

**Canadian Sport Horse Association
P.O. Box 970
Richmond, On
K0A 2Z0**

DO NOT SEND DNA TEST KITS TO MAXXAM OR CLRC

DNA Mane Hair Test - \$60.00 + HST (13%) = \$67.80

VISA _____ MasterCard _____ Number _____ / _____ / _____ / _____ /

Name _____ Expiry Date ____ / ____
As shown on card

Signature _____ Date _____